

QUICK START GUIDE

Register Your Kit | In order for the patient sample to be processed, you'll need to register the kit on the Visible Genomics website.

1. Visit www.VisibleGenomics.io and log into your account
2. Select **"Orders"** and click on **"Create"**
3. Fill out the online form, entering the following details in the fields provided:
 - **Registration code** listed on your kit
 - Name of the doctor requesting the test (or the practice location)
 - All mandatory patient information
 - Test name


Once all of the above information is complete, click **"Create."** You will be required to validate the order by re-entering the last four digits of the kit registration code.



Generate Consent Form

To generate a test consent form, follow these steps:

- Click on **"Actions"** and selecting **"Generate Consent Form"**
- Print out the pre-generated form and follow the instructions. Consent form should be signed by both doctor and patient.
- Scan the signed consent form to your computer
- Upload the signed release form to the Visible Genomics website by clicking on **"Actions"** and selecting **"Upload Signed Consent Form"**

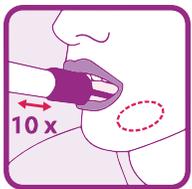

Collect Patient Sample

Collect patient sample by following the instructions on the reverse side of this card.



Enable Tracking

Once your sample has been collected, place it in the pre-paid, pre-addressed return envelope and place it in the mail.

Don't forget!

Once you have mailed the sample to the laboratory, you'll need to confirm this action and enable tracking on the Visible Genomics website:

- Visit www.VisibleGenomics.io
- Click on **"Orders"** and select your existing order
- Click **"Sent to Lab"**


Collection Precautions



Donor should not eat, drink, smoke or chew gum for 30 minutes before collecting saliva sample.


Ensure that the sponge tip does NOT come into contact with any surface prior to collection.

Procedure for saliva sample collection

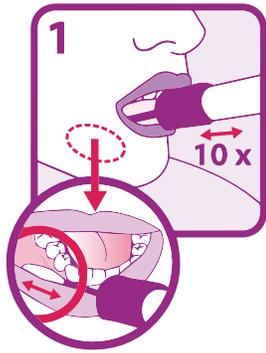

Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close up image) in a back and forth motion. Gently rub the gums 10 times.

If possible, avoid rubbing the teeth.

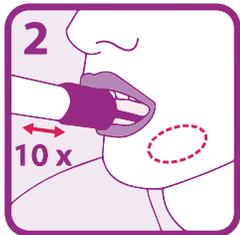


Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.



Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.



Turn the cap upside down, insert the sponge into the tube and close cap tightly.

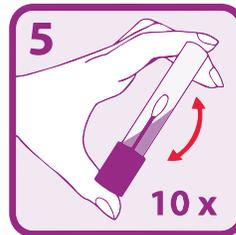

Invert the capped tube and shake vigorously 10 times.

After the sample is collected

Place the collected sample into the enclosed biohazard bag and seal the bag.

Place the sealed biohazard bag in the pre-paid, pre-addressed return envelope provided.

Seal the return envelope and place it in the mail.